

GASTRIC BALLOON



01

SPEEDY PROCESS

The whole process only takes about 30 minutes, the doctor will fill your gastric water balloon with 550ml of saline.



02

NO SURGERY

No need for open and localised surgery, just swallow a capsule.



03

AUTO-DISCHARGE

After 4 months, the water balloon in the stomach is automatically removed from the faeces.

RANGE OF APPLICABLE PEOPLE

1. Aged 18 and above
2. BMI 27 and above
3. Not currently pregnant or breastfeeding
4. Have not had stomach or esophageal surgery

NOTE : Doctor will examine the patient and thoroughly review his medical history to confirm his eligibility.

HOW TO USE

STEP 1

SWALLOW CAPSULE

No need for open and localised surgery just swallow a capsule.

STEP 2

TAKE AN X-RAY

A first X-ray is done to ensure the capsule is placed correctly in your stomach.

STEP 3

FILL BALLOON

The doctor uses the catheter to fill the balloon with 550ml of liquid.

EFFECT OF USE

Average weight loss: 13.5 ± 5.8 kg


Percentage of total weight loss:

$14.2 \pm 5.0\%$

Average BMI Reduction

4.9 ± 2.0 points

Rare Side Effects

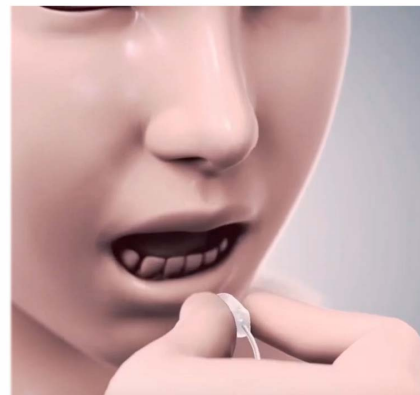

AFTER SALE SERVICE

SIDE EFFECT

It is important to note that the Gastric Balloon may cause adverse events. Some of which may require removal by endoscopy. Rare cases of spontaneous hyperinflation, small intestine obstruction, stomach dilation, oesophagitis, pancreatitis, or stomach perforation have been reported.



INSTRUCTIONS

You must be available for follow-up with your doctor throughout the treatment period, especially if you experience the following symptoms, including but not limited to : persistent nausea, vomiting, dehydration, and/or abdominal pain. If you consult other healthcare professionals, inform them that a gastric balloon is present. We encourage you to stay close to modern emergency care facilities during the time the balloon is in place, in case serious complications arise.